# MOEN®

INS10756 - 2/19

INS10756

#### Please Contact Moen First

For Installation Help, Missing or Replacement Parts  
(USA)  
1-800-BUY-MOEN (1-800-289-6636)  
WWW.MOEN.COM

(Canada)  
1-800-465-6130  
WWW.MOEN.CA

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#### HELPFUL TOOLS

For safety and easy assembly of this product, Moen recommends the use of these helpful tools.

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**WARNING:** Please follow these assembly instructions carefully. Assembler assumes responsibility for proper assembly. Failure to follow correct assembly procedures can result in serious injury.

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

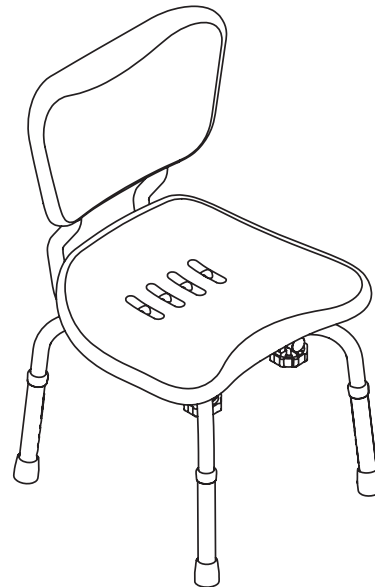
**WARNING:** TO PREVENT SERIOUS OR FATAL INJURY : FOR USE ONLY BY PERSONS WEIGHING LESS THAN 250 POUNDS. Consult a healthcare professional for advice and instructions specific to your needs before use. Contact Moen immediately at 1-800-289-6636 if any components are missing or damaged. Install in accordance with Moen's instructions. Do not modify this product in any way. FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION CONTACT MOEN AT 1-800-289-6636.

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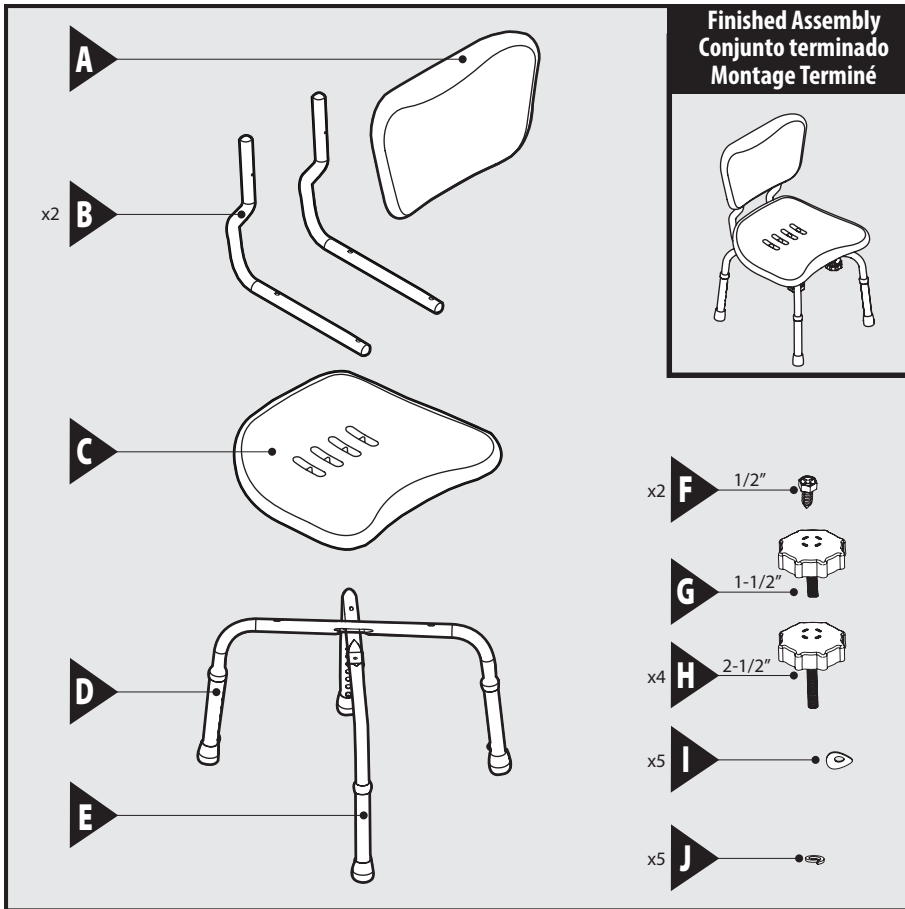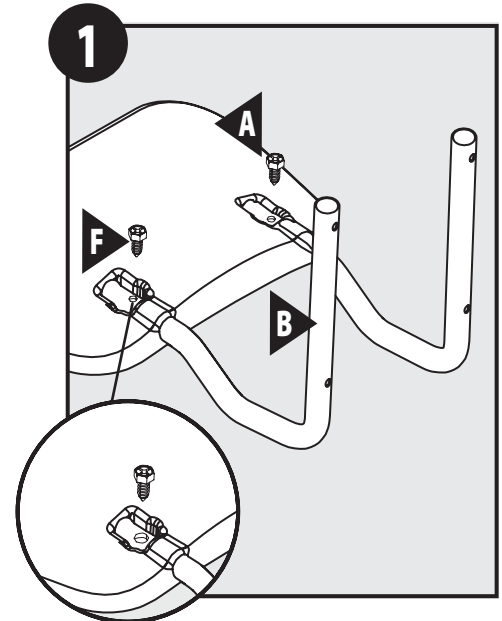Finished Assembly  
Conjunto terminado  
Montage Terminé

1  
Insert the back support tubes (B) into the holes located on rear of seat back (A). Thread 1/2" back support screws (F) into the small holes of the back support tubes (B), screws facing up.

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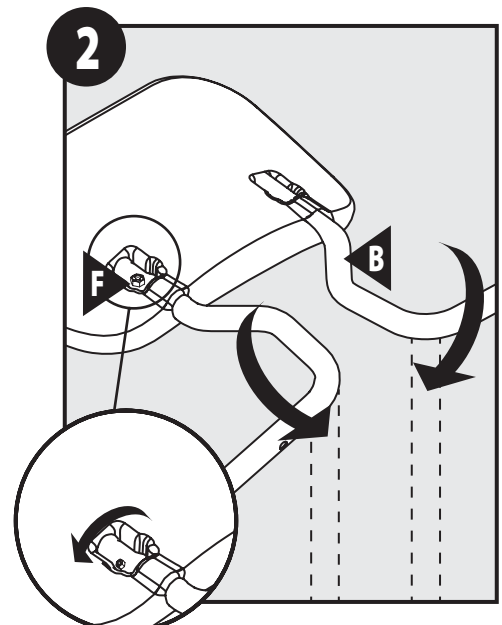
**IMPORTANT:** For stability, chair must be assembled with **ALL** 5 screws.

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
2  
Align back support screws (F) and slot, rotate back support tubes (B) 180° to lock the back support arms in place.

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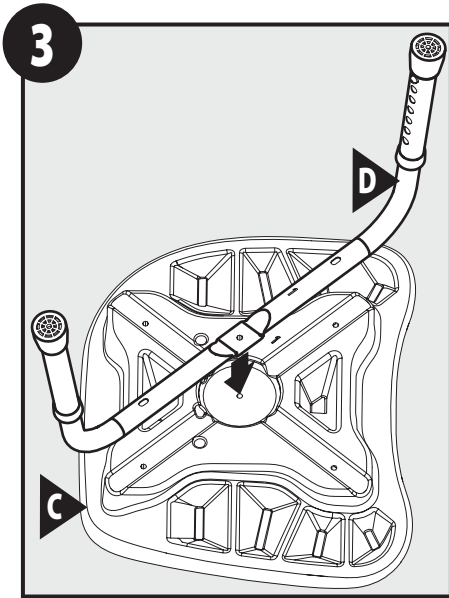
With seat (C) upside-down, place chair leg #1 (D) into groove #1. Make sure the number "1" mark on the seat and leg are next to each other.

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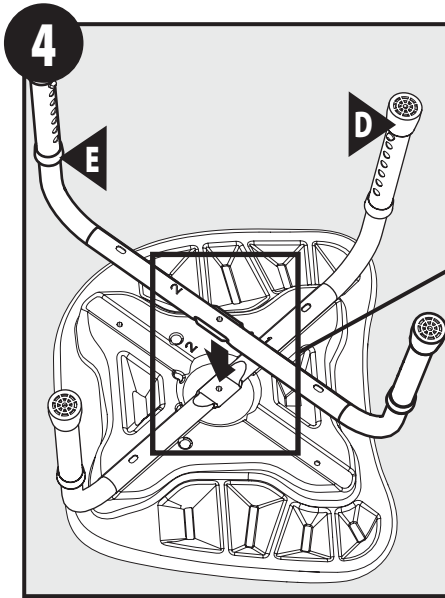
Place chair leg #2 (E) over chair leg #1 (D) so the number "2" marks on the leg and seat are next to each other.

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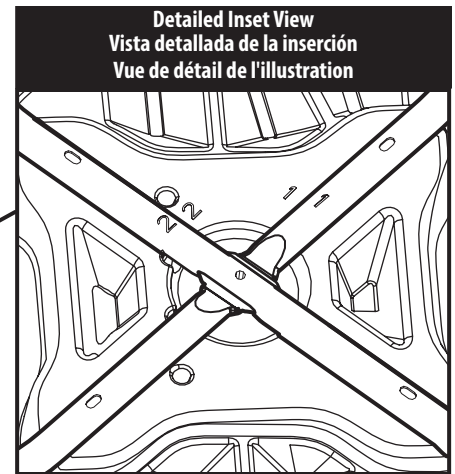
Ensure the numbered legs are properly aligned in their corresponding grooves before proceeding.

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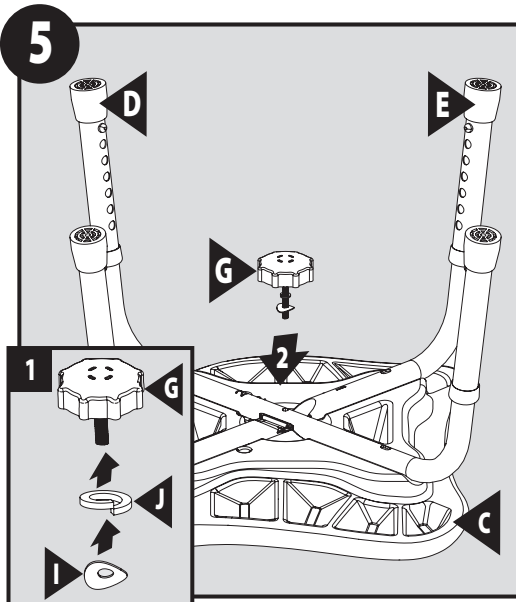
1. Install 1-1/2" hand screw (G) through a lock washer (J) and curved washer (I) as shown.
2. Tighten chair leg #1 (D) and chair leg #2 (E) into center hole in seat (C) by using assembled hand screw (G).

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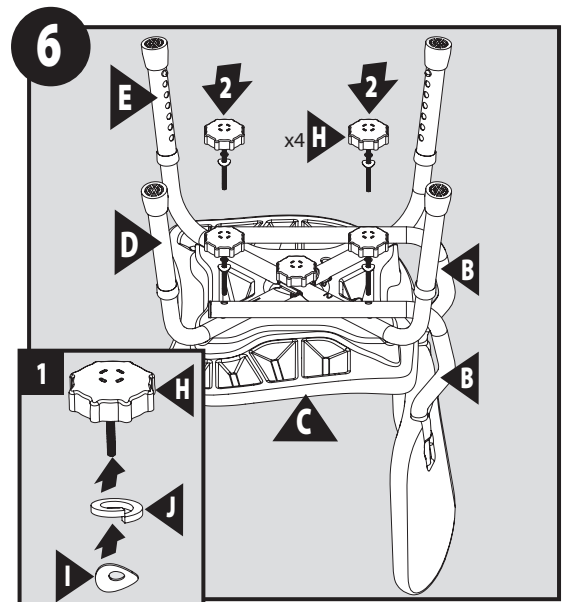1. Install a 2-1/2" hand screws (H) through a lock washer (J) and curved washer (I) as shown.
2. Secure backrest assembly (B) to seat (C) by using 2-1/2" Hand Screws (H) (x4) assembly.

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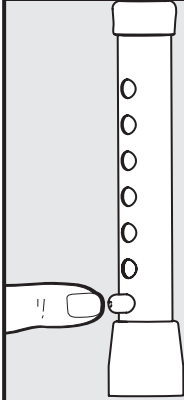
To adjust height, push in button on leg, slide leg to desired height and allow button to pop out and lock leg in place. Repeat for each leg making sure all four are adjusted to the same height.

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## WARRANTY FOR MOEN HOME CARE BATH SAFETY PRODUCTS:

### LIMITED LIFETIME WARRANTY

Moen warrants to the original consumer purchaser for as long as the original consumer purchaser owns their home (the "Warranty Period" for homeowners) if this product fails due to materials or workmanship at any time during the life of the product Moen will replace it free of charge. Purchasers for residential use where the occupant is not the owner of the property (apartments, duplexes, or other multi-residential unit housing) are warranted for a period of 10 years from the original date of purchase (the "Warranty Period" for multifamily owners). Purchasers for industrial, institutional and all other business uses are warranted for a period of 5 years from the original date of purchase (the "Warranty Period" for commercial owners.) Proof of purchase (original sales receipt) from the original consumer purchaser must accompany all warranty claims. Contact Moen at 1-800-289-6636 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. **TO THE EXTENT PERMITTED BY LAW, MOEN DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.**

### LIMITATION OF LIABILITY:

THIS WARRANTY IS YOUR SOLE REMEDY AND MOEN SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL MOEN'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit  
[www.moen.com](http://www.moen.com)

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©2019 Moen Incorporated
U.S.A.
Moen de Mexico, S.A. de C.V.
Moen Inc.